


- Do not attempt to adjust or service the prosthesis, except as advised by your prosthetist.
- Inform your prosthetist if you lose or gain a significant amount of weight.
- Have the prosthesis serviced at the regular intervals specified by the prosthetist.
- Freedom Innovations' feet are manufactured and tested for a particular weight and activity level. Use by an amputee other than the one for whom they were originally manufactured may be dangerous to the amputee, and shall void any written or implied warranty.
- Never attempt to loosen the bolt(s) affixing the pyramid attachment.

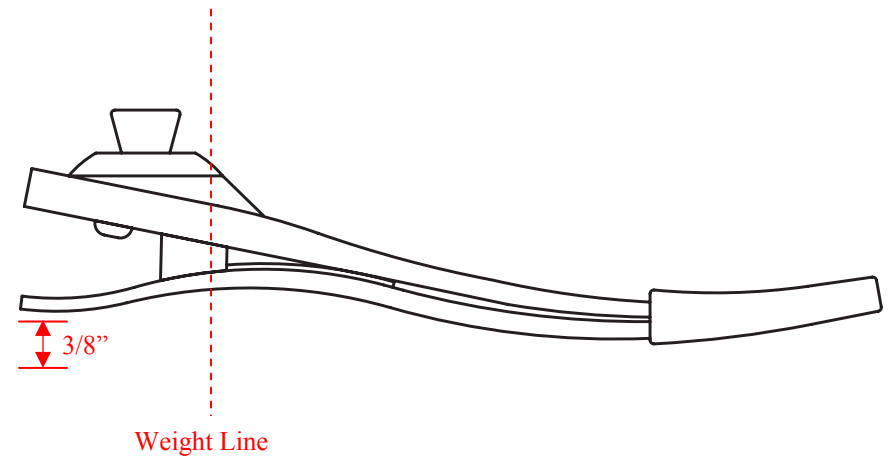

## MAINTENANCE & DISINFECTION

- Freedom Innovations' foot modules should be inspected and serviced every six months, or as advised by the prosthetist.
- The foot may be cleaned and/or disinfected with soap and warm water. If the foot is used in water, or extremely dirty environments, it is advisable to remove the footshell to clean and disinfect the interior of the shell.



## Pacifica™ LP Prosthetic Foot

### Instructions for Use

R-720-073 **Pacifica™ LP, Prosthetist Instructions**, Rev. C

Freedom Innovations, LLC ▪ 30 Fairbanks, Suite 114, Irvine, California 92618  
**Toll Free (888) 818-6777 FAX (949) 672-0084**



**EC REP** Advena Ltd., Hereford, HR4 9DQ UK

## Pacifica™ LP Prosthetic Foot

### BENCH ALIGNMENT

#### Sagittal Plane:

- Introduce the appropriate socket flexion and heel height, according to the amputee's requirements.
- Position the weight line, taken from the center of the socket at the patellar tendon level, so that it falls along the anterior edge of the pylon.

#### Coronal Plane:

- Introduce the appropriate socket adduction/abduction, according to the amputee's requirements.
- Position the weight line, taken from the center of the socket at the patellar tendon level, so that it falls along the midline of the foot module in a neutral M-L position.
- Set appropriate toe-out.

### STATIC ALIGNMENT

- Fit the anatomical foot cover onto the foot module and place into shoe. Use heel wedges to ensure that the weight line falls along the anterior edge of the pylon with shoe in place.
- Establish the correct height of the prosthesis by having the amputee stand with feet at shoulder width, and equal weight bilaterally.
- Check socket flexion, load line position and toe-out. Correct as necessary.

### DYNAMIC ALIGNMENT

#### Gait:

- The function of the foot may be optimized by modifying the alignment of the socket relative to the foot and by altering heel stiffness. The heel stores energy at heel strike and progressively releases it at midstance. Careful attention to alignment will ensure optimal energy return, and improve control of the prosthesis.
- As the amputee walks, check for smoothness of gait and even ground contact at all times.
- Optimize the alignment by progressively moving the weight line anteriorly over the foot to increase heel stiffness and observing the heel to toe function.

- The heel is too soft if the foot comes to foot flat too early or the amputee feels like they are sinking or trying to climb up over the toe. The toe will feel too stiff and the knee may hyperextend. Shift the socket anteriorly and/or use foam wedges to stiffen the heel, and soften the toe.
- The heel is too hard if the amputee's gait exhibits rapid heel to toe movement and they have difficulty in controlling the prosthesis. The toe may feel sluggish with minimal energy return and the knee may buckle. Shift the socket posteriorly, or remove heel stiffening foam to alleviate this problem.
- Use Loctite T 242 (not provided), and torque pyramid adjustment screws to the manufacturer's recommendations. All screws should be re-torqued after dynamic alignment of the prosthesis.
- Freedom Innovations foot modules are manufactured to fit industry standard pyramids and receivers. It is the prosthetist's responsibility to choose and fabricate properly fitting connectors.

#### Altering Heel Stiffness:

- In most cases, stiffening the heel is best achieved by moving the weight line anteriorly or dorsiflexing the foot. However, the heel action may also be stiffened, by affixing the heel stiffening foam between the heel and toe portions of the foot module.
- During dynamic alignment, hold the heel stiffening foam in place by applying the double-sided adhesive strip to the bottom surface of the foam. **Moving the foam forward will soften the heel of the foot. Moving the foam towards the heel will stiffen the foot.**
- After the proper foam location has been selected, remove the temporary adhesive strip, lightly scuff the area to be glued, clean the mating surface with a solvent and apply with a permanent adhesive such as Barge Cement (not provided). Do not glue the upper portion of the stiffening foam to the shank of the foot, as this will affect foot performance and lead to eventual failure of the stiffening foam.

### WARNINGS & CONTRAINDICATION

No specific warnings and/or contraindications are specified for this device, although they should be used for the assembled prosthesis. These include, but are not limited to:

- Discontinue use, and consult your physician or prosthetist if the prosthesis causes pain, or injures you in any way.