

# Freedom Agilix™

## Maximum shock value

- **Vertical shock absorption** and carbon fiber flexibility optimized for low to moderate impact K3 ambulators; reduces socket shear forces and improves overall comfort
- **Multi-axial function** increases ground compliance and stability; streamlined, split-toe design results in improved long-term durability, saving time and money
- **Full length dynamic heel** attached at the toe ensures a seamless roll-over, for superior comfort and amore symmetrical gait
- **Lower profile** and **lighter weight** than the competitive alternative, allowing users to do more and feel less tired at the end of the day
- Rated to 166 kg (365 lbs)
- 36-month warranty



Freedom Agilix (F15)



Vertical Shock



Split Toe



Multi Axial



Sandal Toe

**Product ID:** F15

**Sizes:** 22-31 cm


Sandal-toe available for sizes 22-28 cm

**User Weight Rating:** 166 kg (365 lbs)

**Heel Height:** 10 mm (3/8")

**Connector:** Male pyramid

**Warranty:** Foot module 36 months,  
Cosmetic foot shell 6 months

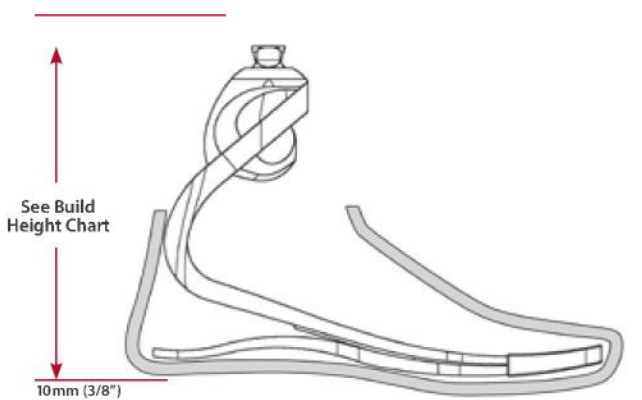

Note: Build height measurements represent a fully assembled unit.

## Accessories

- Foot Shell Removal Tool: (ACC-00-103)
- Spectra Sock: (ACC-00-101xx) 12", 14", 16"
- Stiffening Bumpers:
  - 22-25cm: (KIT-00-11363-00)
  - 26-28cm: (KIT-00-11364-00)
  - 29-31 cm: (KIT-00-11365-00)

## Build Height & Weight

Size (cm)	Height (in / mm)	Weight (oz / g)
22-25	5¾ / 143	19 / 553
26-28	6⅓ / 161	24 / 609
29-31	6¾ / 172	27 / 772

Product weight includes foot shell. Measurements are approximate and will vary due to category selection and manufacturing variability.

## Stiffness Category Selection

Weight (lbs)	Impact Level		Weight (kg)
	Low	Moderate	
100-115	1	1	44-52
116-130	1	2	53-59
131-150	2	3	60-68
151-170	3	4	69-77
171-195	4	5	78-88
196-220	5	6	89-100
221-255	6	7	101-116
256-285	7	8	117-130
286-325	8	9	131-147
326-365	9		148-166

## Foot Module Part Number

**F15-00-0**  **A**  -

Category: 1-9      Size: 22-31      Toe Configuration: RU-regular toe, universal; SR-sandal toe, right; SL-sandal toe, left

## Foot Shell Part Number

**FTC-**   -    **4-**

Generation: 2-3      Profile: M-mid, H-high      Cap: 0=yes, 1=no      Size: 22-31      Skin Tone: L-light, M-medium, D-dark      Toe: R-regular, S-sandal      Side: L-left, R-right

See the Cosmetic Foot Shell page for more detailed ordering instructions.