

Highlander® & Highlander® MAX

Proven technology for dependable performance

- Offers an ideal blend of stability and action, often preferred by transfemoral amputees
- Split toe provides excellent inversion/eversion
- Custom Highlander MAX available for users weighing up to 226 kg (500 lbs), sold with optional heavy duty 34 mm component kit
- Standard Highlander rated to 166 kg (365 lbs)
- 36-month warranty



Split Toe



Sandal Toe

Highlander (FS3)

Product ID: FS3

Sizes: 22-31 cm

Sandal-toe available for sizes 22-28 cm

User Weight Rating: 166 kg (365 lbs)

Highlander MAX: Custom order* rated to 226 kg (500 lbs),

34 mm pyramid, 26-31 cm


Heel Height: 10 mm (3/8")

Connector: Male pyramid

Warranty: Foot module 36 months,

Cosmetic foot shell 6 months

*Custom orders: 7-10 day lead time, no trial period

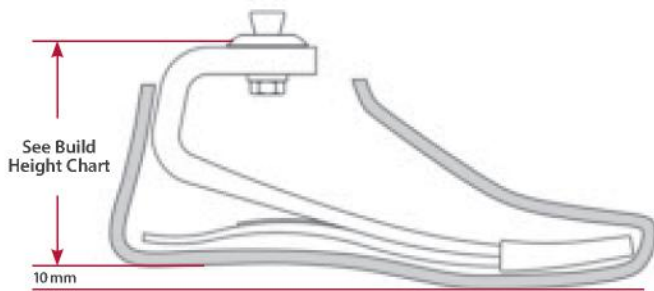

Note: Build height measurements represent a fully assembled unit.

Accessories

- 34 mm Component Kit: (KIT-00-202), rated to 226 kg (500 lbs); kit includes pylon, two female tube clamps, and a 4-ear connector
 - Tube Clamp, 34 mm: (ACC-00-20201-00)
 - 4-Ear Connector: (ACC-00-20202-00)
 - Pylon, 34 mm: (ACC-00-20203-00)
- Foot Shell Removal Tool: (ACC-00-103)
- Spectra Sock: (ACC-00-101xx) 12", 14", 16"
- Stiffening Bumpers:
 - 22-25cm: (KIT-00-11303-00)
 - 26-28cm: (KIT-00-11304-00)
 - 29-31cm: (KIT-00-11305-00)

Build Height & Weight

Size (cm)	Height (in / mm)	Weight (oz / g)
22-25	3 ⁷ / ₈ / 98	17 / 470
26-28	4 ¹ / ₄ / 108	21 / 590
29-31	4 ³ / ₄ / 121	26 / 750

Product weight includes foot shell. Measurements are approximate and will vary due to category selection and manufacturing variability.

Stiffness Category Selection

Weight (lbs)	Impact Level			Weight (kg)
	Low	Moderate	High	
100-115	1	1	2	44-52
116-130	1	2	3	53-59
131-150	2	3	4	60-68
151-170	3	4	5	69-77
171-195	4	5	6	78-88
196-220	5	6	7	89-100
221-255	6	7	8	101-116
256-285	7	8	9	117-130
286-325	8	9	10	131-147
326-365	9	10	11	148-166
366-420	10			167-190
421-480	11			191-218
481-500	12			219-227

Foot Module Part Number

FS3- - **A** -

Module —
 00 (Standard categories)
 HS (Highlander MAX)

Category
 01-09;
 10-12

Size
 22-31

Toe Configuration
 RU-Regular toe,
 Universal
 SR-Sandal toe, Right
 SL-Sandal toe, Left

Foot Shell Part Number

FTC- - **4-**

Generation —
 2-3

Profile
 M-mid
 H-high

Cap
 0=yes
 1=no

Size
 22-31

Skin Tone
 L-light
 M-medium
 D-dark

Toe —
 R-Regular
 S-Sandal

Side —
 L-left
 R-right

See the Cosmetic Foot Shell page for more detailed ordering instructions.