

August 13, 2015

## ACCESSORIES

P/N	Product Name	Item	Torque Spec
ACC-EX-12300-00	Extended Tube Clamp, 34mm Female	Alignment setscrew	15Nm / 133in-lb
		Pylon clamp cap screw	15Nm / 133in-lb
ACC-00-12500-00	30mm Female Tube Clamp	Alignment setscrew	15Nm / 133in-lb
		Pylon clamp cap screw	7Nm / 62in-lb
ACC-00-12600-00	30mm Male Tube Clamp	Pylon clamp cap screw	7Nm / 62in-lb
ACC-00-12700-00	30 x 100mm Female Tube	Alignment setscrew	15Nm / 133in-lb
ACC-00-12800-00	30 x 400mm Female Tube	Alignment setscrew	15Nm / 133in-lb
ACC-00-12900-00	34mm Female Tube Clamp	Alignment setscrew	15Nm / 133in-lb
		Pylon clamp cap screw	15Nm / 133in-lb
ACC-00-13200-00	4-Hole Female Pyramid	Alignment setscrew	15Nm / 133in-lb
		Mountingscrew	10Nm / 88in-lb
ACC-00-13500-00	4-Hole Rotating Male Pyramid	Mountingscrew	10Nm / 88in-lb
		Rotation clamp screw	10Nm / 88in-lb
ACC-00-13600-00	4-Hole Rotating Female Pyramid	Alignment set screw	15Nm / 133in-lb
		Mountingscrew	10Nm / 88in-lb
		Rotation clamp screw	10Nm / 88in-lb
ACC-00-13700-00	4-Hole Rotating Pyramid Base	Mountingscrew	10Nm / 88in-lb
		Rotation clamp screw	10Nm / 88in-lb
ACC-00-13800-00	3-Prong Rotating Pyramid	Rotation clamp screw	10Nm / 88in-lb
ACC-00-14000-00	Rotating Female Pyramid	Alignment setscrew	10Nm / 88in-lb
ACC-00-16100-00	4-Hole Lamination Base	Mountingscrew	5Nm / 44in-lb
ACC-00-17032-00	32mm Dual Female Adaptor	Alignment setscrew	15Nm / 133in-lb
ACC-00-17045-00	45mm Dual Female Adaptor	Alignment setscrew	15Nm / 133in-lb
ACC-00-17060-00	60mm Dual Female Adaptor	Alignment setscrew	15Nm / 133in-lb

<b>ACC-00-17075-00</b>	75mm Dual Female Adaptor	Alignment setscrew	15Nm / 133in-lb
<b>ACC-00-19600-00</b>	34mm Male Tube Clamp	Pylon clamp cap screw	15Nm / 133in-lb
<b>ACC-00-20000-00</b>	Offset Tube Clamp, 34mm Female	Alignment setscrew	15Nm / 133in-lb
		Pylon clamp cap screw	15Nm / 133in-lb
<b>ACC-00-20201-00</b>	34mm Female Tube Clamp	Alignment setscrew	20Nm / 177in-lb
		Pylon clamp cap screw	12Nm / 106in-lb

# FOOT PRODUCTS

P/N	Product Name	Item	Torque Spec
<b>F10</b>	Dynadapt	F10 ConnectorBolts	30 ft-lbs & Loctite
<b>F15</b>	Agilix	M12 Connector Bolt (Cat 1-9)	45-50 ft-lbs & Loctite
<b>FS1</b>	Sierra 19-21cm	M10 Connector Bolt (Cat 1-6)	40-45 ft-lbs & Loctite
		M12 Connector Bolt (Cat 7-9)	45-50 ft-lbs & Loctite
	Sierra 22-31 cm	M10 Connector Bolt (22-25)	40-45 ft-lbs & Loctite
		M12 Connector Bolt (26-31)	45-50 ft-lbs & Loctite
<b>FS3</b>	Defender 16-18cm	M8 Connector Bolt (Cat 1P-1A)	16-18 ft-lbs & Loctite
		M10 Connector Bolt (Cat 2A-4A)	40-45 ft-lbs & Loctite
	Defender 19-21cm	M8 Connector Bolt (Cat 1P-1A)	16-18 ft-lbs & Loctite
		M10 Connector Bolt (Cat 2A-4A)	40-45 ft-lbs & Loctite
<b>FS3</b>	Highlander 22-25 cm	M10 Connector Bolt (Cat 1-6)	40-45 ft-lbs & Loctite
		M12 Connector Bolt (Cat 7-9)	45-50 ft-lbs & Loctite
	Highlander 26-28 cm	M10 Connector Bolt (Cat 1-4)	40-45 ft-lbs & Loctite
		M12 Connector Bolt (Cat 5-9)	45-50 ft-lbs & Loctite
	Highlander 29-31 cm	M12 Connector Bolt (Cat 1-9)	45-50 ft-lbs & Loctite
<b>FS3</b>	Highlander MAX	M12 Connector Bolt (Cat 10-13)	45-50 ft-lbs & Loctite
<b>FS4</b>	Pacifica LP 22-25 cm	M8 Pyramid Bolts (Cat 1-6)	15 ft-lbs & Loctite
		M10 Pyramid Bolts (Cat 7-9)	25 ft-lbs & Loctite
	Pacifica LP 26-28 cm	M8 Pyramid Bolts (Cat 1-4)	15 ft-lbs & Loctite
		M10 Pyramid Bolts (Cat 5-9)	25 ft-lbs & Loctite
	Pacifica LP 29-31 cm	M10 Pyramid Bolts (Cat 1-9)	25 ft-lbs & Loctite
<b>FS5</b>	Thrive	M12 Connector Bolt (Cat 1-9)	45-50 ft-lbs & Loctite
<b>FS6</b>	Nitro	M10 Connector Bolt (Cat 1-4)	40-45 ft-lbs & Loctite
		M12 Connector Bolt (Cat 5-9)	45-50 ft-lbs & Loctite
<b>FS9</b>	Nitro LP	M10 Connector Bolt (Cat 1-4)	40-45 ft-lbs & Loctite
		M12 Connector Bolt (Cat 5-9)	45-50 ft-lbs & Loctite
<b>FX6/FX7</b>	Catapult	M12 Connector Bolt	45-50 ft-lbs & Loctite
<b>LP2</b>	LP Symes	M10 Bolts	25ft-lbs

<b>LP2-W2</b>	Swim	M10 Bolts	25 ft-lbs & Loctite
<b>R11</b>	Renegade AT Mold 22-31 cm	M12 Connector Bolt	45-50 ft-lbs & Loctite
<b>R16</b>	Renegade LP-AT 22-25 cm	M10 Connector Bolt (Cat 1-6)	40-45 ft-lbs & Loctite
		M12 Connector Bolt (Cat 7-9)	45-50 ft-lbs & Loctite
	Renegade LP-AT 26-28 cm	M10 Connector Bolt (Cat 1-4)	40-45 ft-lbs & Loctite
		M12 Connector Bolt (Cat 5-9)	45-50 ft-lbs & Loctite
	Renegade LP-AT) 29-31 cm	M12 Connector Bolt (Cat 1-9)	45-50 ft-lbs & Loctite
<b>ROM</b>	Kinterra Foot/Ankle	ROM Platform Bolt (M8)	17 ft-lbs & Loctite
<b>RS1</b>	Renegade 19-21cm (Regular toe only)	M10 Connector Bolt (Cat 1-6)	40-45 ft-lbs & Loctite
		M12 Connector Bolt (Cat 7-9)	45-50 ft-lbs & Loctite
	Renegade Mold 22-31 cm (Regular & Sandal)	M10 Connector Bolt (22-25)	40-45 ft-lbs & Loctite
		M12 Connector Bolt (26-31)	45-50 ft-lbs & Loctite
<b>RS2-P1</b>	Runway Push Button	Slide Bolts (Cat 1-6)	17 ft-lbs & Loctite
<b>RS2-K1</b>	Runway HX	Slide Bolts (Cat 1-6)	17 ft-lbs & Loctite
<b>RS6</b>	Renegade LP 22-25 cm (Regular & Sandal)	M10 Connector Bolt (Cat 1-6)	40-45 ft-lbs & Loctite
		M12 Connector Bolt (Cat 7-9)	45-50 ft-lbs & Loctite
	Renegade LP 26-28 cm (Regular & Sandal)	M10 Connector Bolt (Cat 1-4)	40-45 ft-lbs & Loctite
		M12 Connector Bolt (Cat 5-9)	45-50 ft-lbs & Loctite
	Renegade LP 29-31 cm (Regular toe only)	M12 Connector Bolt (Cat 1-9)	45-50 ft-lbs & Loctite
<b>SKI</b>	Ski Foot	M12 Cup Bolts	45-50 ft-lbs & Loctite
		M8 Toe Bolts	25-30 ft-lbs & Loctite
<b>VS1</b>	Senator	M10 Connector Bolt (Cat 1-6)	40-45 ft-lbs & Loctite
<b>VS2</b>	Promenade	M10 Connector Bolt (Cat 1-5)	40-45 ft-lbs & Loctite