



FREEDOM
KINNEX™
MICROPROCESSOR ANKLE/FOOT SYSTEM

CONNECT WITH BREAKTHROUGH TECHNOLOGY.

Stay Grounded, Feel Connected.

Kinnex™ integrates the world's fastest responding microprocessor ankle technology and a carbon fiber foot to provide users with a uniquely stable and natural walking experience. Whether on flat, angled, smooth or uneven terrain, users will enjoy the heightened confidence that comes from staying firmly grounded and feeling connected with every step — in both wet and dry environments!

- 1 RapidResponse™ Valve Technology**
Adapt to changes in slope within the very first step.
- 2 Sensor-controlled ankle movement**
Sensors instantly adjust stiffness and rotation based on ground characteristics with each step.
- 3 30 degrees of ankle articulation**
The expanded ankle articulation creates a “flexibility” that keeps the ankle and foot positioned in the ideal manner in relation to the ground.
- 4 Dynamic lock**
This feature allows the user to be able to stand on the prosthesis without the ankle shifting back and forth.
- 5 User Settings – Hi Tech and Low Tech**
Use our phone app to monitor battery status, make heel adjustments, select shoes and fine tune performance and comfort settings.
- 6 Apparel Assist Mode**
When activated the foot will lock at an angle that makes it easier to change clothes or footwear.
- 7 A manual lock**
This convenient feature allows to quickly and easily lock out the ankle.
- 8 Freedom Carbon Fiber**
Carbon fiber has the best clinical outcomes due to its energy return properties, which improves performance.

Improved Mobility

- A More Natural Gait
- Stability on Slopes
- Natural Motion



FREEDOM
INNOVATIONS



Patient Selection Guide

1

Verify User's Measurements

- Patient weight
- Foot Size (24-30cm)
- Clearance requirements (*see build height chart*)
- Right or Left
- Sandal Toe*/ Standard Toe

Foot Size	Build Height
24-25cm	7.2" / 184mm
26-28cm	7.3" / 186mm
29-30cm	7.4" / 187mm

*Available Sandal Toe Sizes: 24cm-28cm

2

Select Stiffness Category

Weight (lbs)	Impact Level		Weight (kg)
	Low	Moderate	
100-115	1	1	44-52
116-130	1	2	53-59
131-150	2	3	60-68
151-170	3	4	69-77
171-195	4	5	78-88
196-220	5	6	89-100
221-255	6	7	101-116
256-275	7		117-125

3

Complete Foot Module Part Number

F14-00-0 **A** -

Category
1-7
Size
24-30
Toe Configuration
 RL-Regular toe,
 Unilateral
 SL-Strut at toe, Right
 SL-Strut at toe, Left

www.freedom-innovations.com

Toll Free: 888-818-6777

Global Headquarters

3 Morgan Irvine CA 92618 USA